SOPIRA® Carpule® Syringe Aspirating

| | | |
|-----------|---------------------------|----|
| DE | Gebrauchsanweisung..... | 2 |
| GB | Instructions for use..... | 4 |
| FR | Mode d'emploi..... | 6 |
| ES | Instrucciones de uso..... | 8 |
| IT | Istruzioni per l'uso..... | 10 |
| PT | Instruções de uso..... | 12 |
| NL | Gebruiksaanwijzing..... | 14 |
| SE | Bruksanvisning..... | 16 |
| DK | Brugervejledning..... | 18 |
| NO | Bruksanvisning..... | 20 |
| FI | Käyttöohjeet..... | 22 |
| GR | Οδηγίες Χρήσης..... | 24 |

Heraeus

99000587/00 - 01.2009

Heraeus Kulzer GmbH
Grüner Weg 11
63450 Hanau (Germany)

info.dent@heraeus.com
www.heraeus-dental.com

Made in Germany

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
Instructions for use **GB**

Aspirating dental cartridge syringe


In combination with SOPIRA Carpule disposable needles the SOPIRA Carpule syringe provides a universal, easy-to-use injection system.

- All standardised cartridges (1.8 ml) with rubber plunger (solid as well as recessed) can be used with the SOPIRA Carpule Syringe.
- The rotatable cartridge holder permits perfect view of the cartridge in every situation, especially when controlling the aspiration process and the volume of the injected solution.
- The chromium-plated SOPIRA Carpule Syringe is easily to dismantle, allowing easy cleaning and sterilization.


Instructions for use: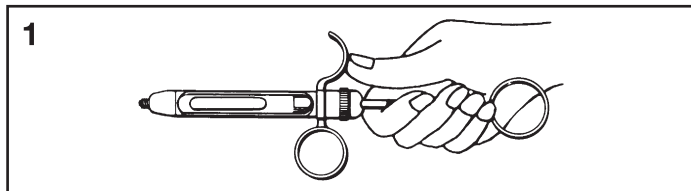


Withdraw the piston rod completely.

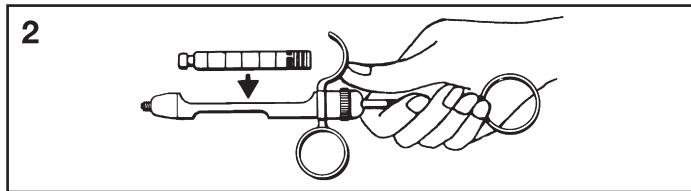Load the cartridge.

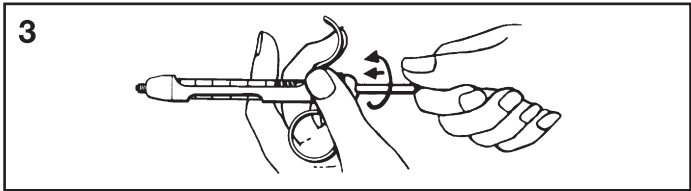
Screw the spiral hooks clockwise into the rubber plunger applying light pressure.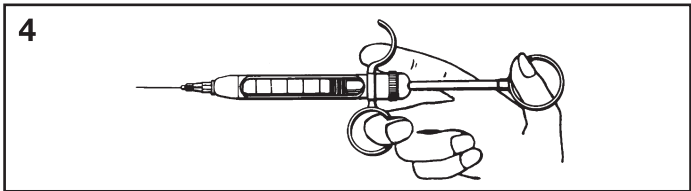

Fix the needle:

Ensure that the cartridge end of the needle pierces the centre of the rubber membrane. It is advisable to insert the needle just prior to injecting. Expel a few drops of the solution. The syringe is now ready for use.

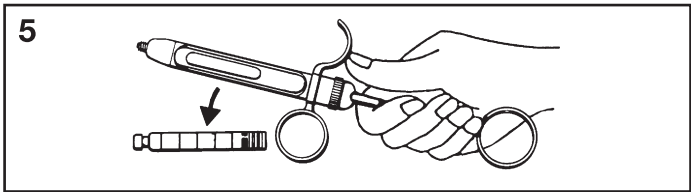After injection:

Detach the needle and remove the spiral hooks from the rubber plunger. Fully retract the piston rod and remove the cartridge.

Sterilisation:

After use, the SOPIRA Carpule Syringe can be sterilised in your usual equipment. Before sterilization the syringe should be cleaned with appropriate detergents.

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GB Our technical advice, whether verbal, in writing or by way of trials, is given in good faith but without warranty, and this also applies where proprietary rights of third parties are involved. It does not release you from the obligation to test the products supplied by us as and to their suitability for the intended processes and uses. The application, use and processing of the products are beyond our control, and therefore, entirely your own responsibility. Should, in spite of this, liability be established for any damage, it will be limited to the value of the goods delivered by us and used by you. We will, of course, provide consistent quality of our products within the scope of our General Conditions of Sale and Delivery.

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